

Magsella chair
Product Manual



Thank you for purchasing our Magsella chair.

This manual aims to help you understand the product details, use it safely and correctly, and obtain a satisfactory experience. Furthermore, you can obtain information about product quality, troubleshooting, and after-sales service.

Note:

Before installing, using, or debugging this product, please carefully read and understand the contents of this product manual to ensure correct use of the product, and to ensure the safety of the user and operator.

! This product manual introduces the product according to the most complete configuration, so some content may not be applicable to the product you purchased.

! If you have any questions, please contact our company.

Table of Contents

I	Introduction and Uses	3
II	Product Parameters	4
III	Precautions	5
IV	Use and Installation	7
V	Common Problems	10
VI	Troubleshooting	11
VII	Maintenance and Repair	12
VIII	Warranty Terms	13

The "**Magsella chair**" revolutionizes traditional treatments for urinary incontinence. Through the revolutionary "High pulse" technology—a high-intensity focused electromagnetic field—it stimulates motor neurons to control the pelvic floor muscles, inducing repetitive hypercontraction movements to rebuild the strength and endurance of the pelvic floor muscles and urinary system.

II

Product Parameters



【Product Name】	Magsella chair
【Power Consumption】	2.1KVA
【Operating Voltage】	110v/220v/50/60Hz
【Magnetic Energy】	0-3.4T
【Pulse Mode】	Biphase Wave (cm)
【Dimensions】	
【Packaging Weight】	

Before use, please read these precautions carefully to help you use this product safely and correctly.

Absolutely Contraindicated Groups:

!X 1. It is prohibited to use metal implants in the body, or areas with other artificial fillers.

!X 2. It is prohibited for pregnant women and lactating women.

!X 3. It is forbidden for patients with major diseases such as malignant tumors and cancer.

!X 4. Patients with heart disease, arrhythmia, epilepsy, severe hypertension, and severe diabetes are prohibited from doing this.

Groups at Risk:

!!! 1. Women during menstruation should use with caution, as it may cause abdominal cramps.

!!! 2. Patients who have undergone major surgery should consult their doctor based on their recovery progress before using this product.

!!! 3. People with open wounds should avoid the wound area.

!!! 4. Do not stimulate the abdomen when full.

Product Safety Precautions:


- ! ! 1. If you experience any abnormal sensations during use, please stop using it immediately and consult a professional.
- ! ! 2. Before use, you must remove any metal objects worn near the stimulation area.
- ! ! 3. Do not touch the power plug after plugging it in, and do not pull the power cord forcefully.
- ! ! 4. Do not touch the power cord or power plug with wet hands.
- ! ! 5. Do not forcefully impact the instrument or the area around the power supply.
- ! ! 6. Do not use the instrument at high temperatures, and do not place heating devices near the instrument.
- ! ! 7. If the instrument emits smoke or an unusual odor during use, please disconnect the power and consult after-sales service.


Product Operation Precautions:

- ! ! 1. Do not operate the touch control screen with wet hands.
- ! ! 2. Do not forcefully pull the touch control screen cable.
- ! ! 3. When using the instrument, first lower the intensity, and then gradually adjust it to the appropriate intensity.
- ! ! 4. After each use, wait for ten minutes before turning off the power.
- ! ! 5. Do not disassemble the internal components of the product yourself to avoid possible product malfunction;
- ! ! 6. If the instrument malfunctions, please check the instruction manual again before operating; if the malfunction persists, do not disassemble the instrument yourself, but contact after-sales service for consultation.
- ! ! 7. When the instrument is not in use for a long time, please unplug the power cord.

IV

Use and Installation

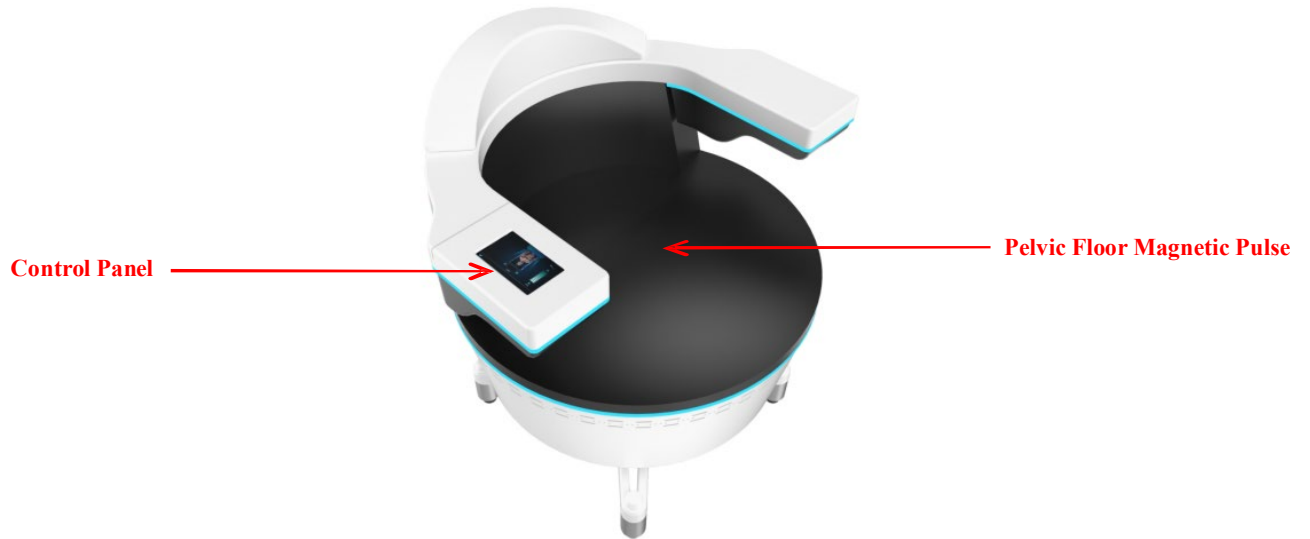


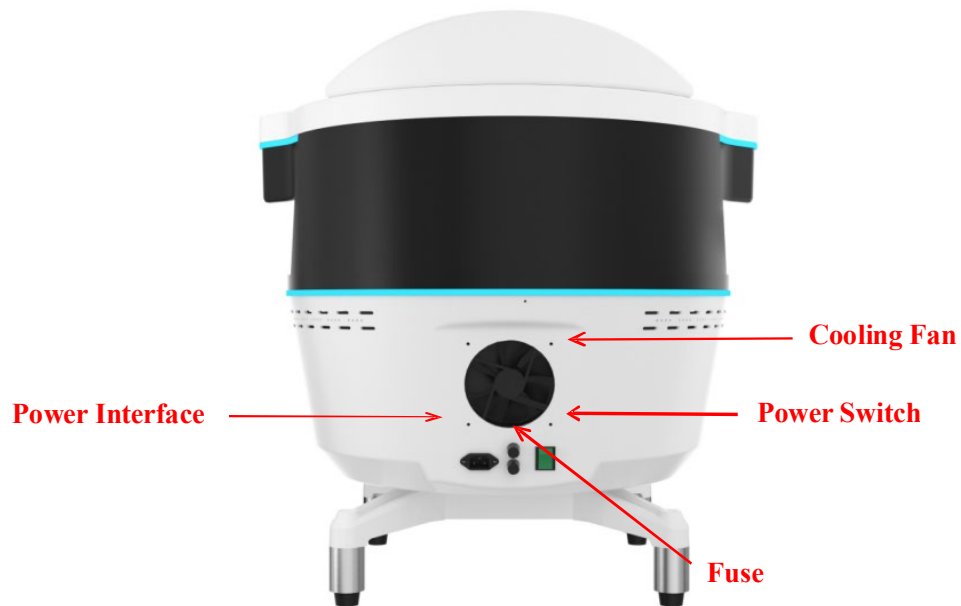

IV


Use and Installation

- Before use, select according to gender and part:

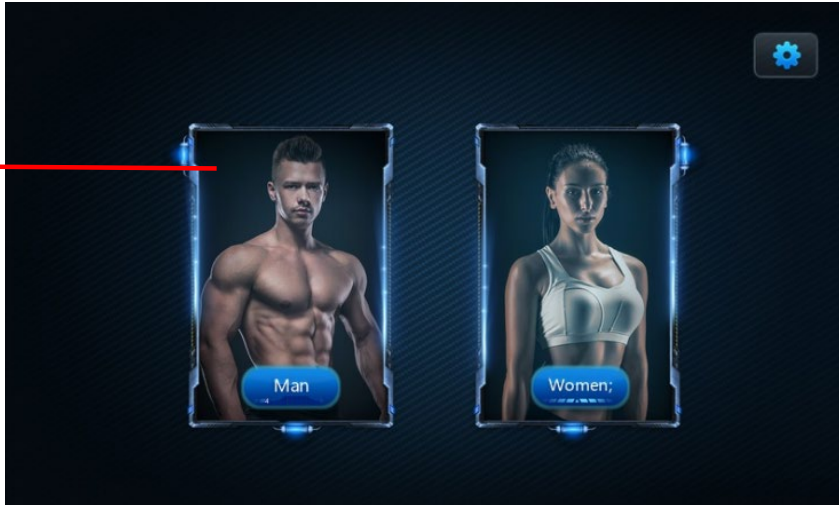


**Male / Female
Mode Selection**

Muscle Building and Shaping

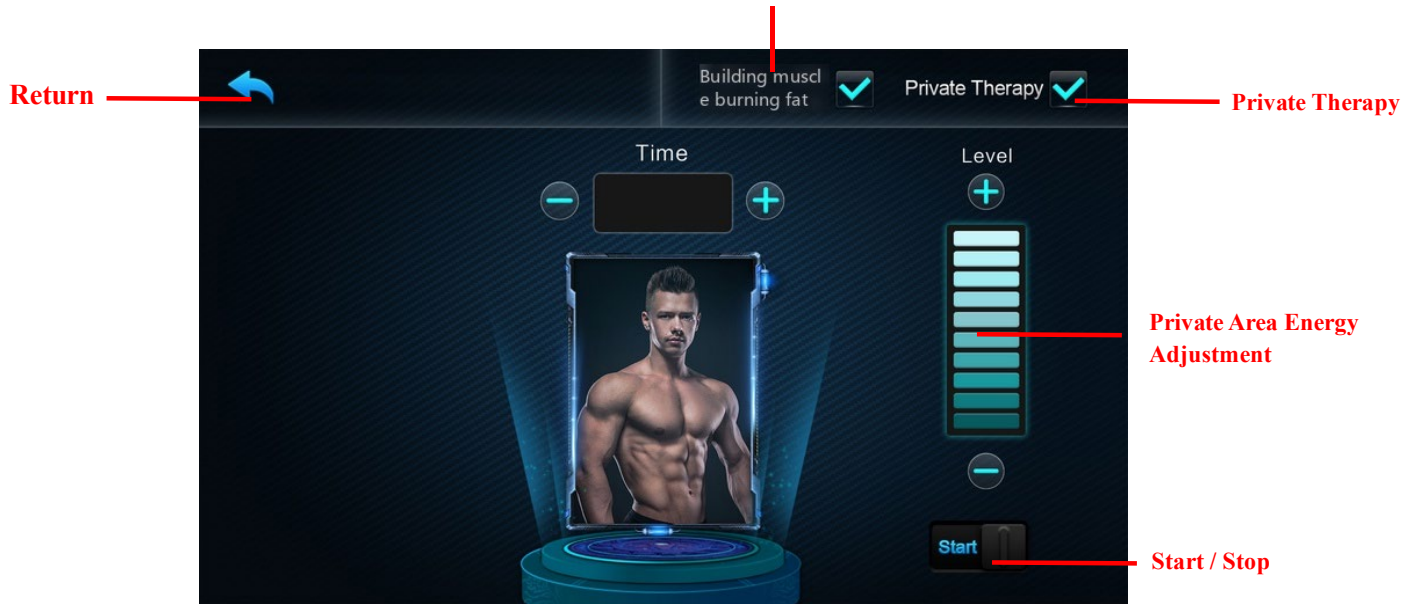

Muscle Building and Buttock Lifting

Return

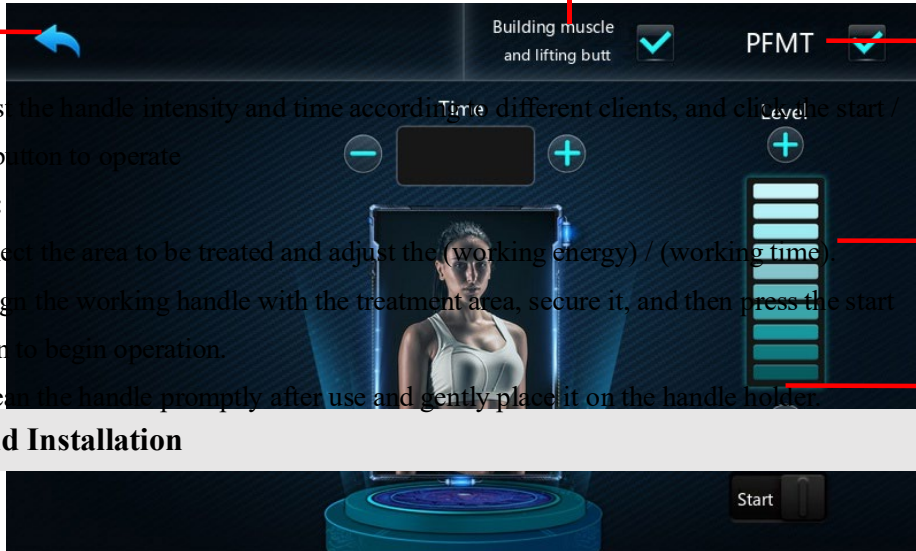

Pelvic Floor Muscle Training

Adjust the handle intensity and time according to different clients, and click the start / stop button to operate.

Note:

1. Select the area to be treated and adjust the (working energy) / (working time).
2. Align the working handle with the treatment area, secure it, and then press the start button to begin operation.
3. Clean the handle promptly after use and gently place it on the handle holder.

Private Area Energy Adjustment

Start / Stop

IV

Use and Installation

1. Will I experience discomfort while using the Magsella chair?

A: During use, the energy intensity should be gradually increased from low to high to allow the user time to adapt. For users with sensitive bodies, give them time to adjust to the energy level.

If you experience palpitations or unbearable muscle tension in the area being used, immediately reduce the energy or stop using it.

2. How long does it take to see results with the Magsella chair?

A: Usually, you can feel the effect immediately after use, with the best effect lasting 1-2 weeks after use. Using it in cycles (4-6 cycles) can maintain a long-term effect.

3. What is the cycle plan for the Magsella chair?

A: According to research, it is used on local muscles, 20 minutes each time, 4-5 times a week, for two weeks as one cycle, lasting for two weeks. However, some users may need more cycles to achieve the best effect. It is recommended to perform a cycle every 1-2 months to maintain the best effect.

4. Is the energy penetration depth safe? Inside the body?

A: Pulsed magnetic technology has been widely used, and its safety has been proven by dozens of studies and a large number of users. The only tissue that reacts to magnetic energy is the moving muscles, so it has no effect on other internal tissues of the body.

5. Who is not suitable for the Magsella chair?

A: It is a widely applicable product suitable for the vast majority of people. However, it is necessary to avoid high-risk groups according to the precautions.

VI**Troubleshooting**

Serial Number	Code Encountered	Possible Causes	Solution
1	E01	Functional Box Temperature Alarm	Auto stop, restart after cooling down
2	E03	Coil Temperature Alarm	Auto stop, restart after cooling down

VII

Maintenance and Repair

1 Maintenance

1.1 Daily Inspection

Management personnel must regularly inspect instruments and components.

Before using the instrument, check if the power cord is damaged. If damaged, stop using the instrument.

1.2 Cleaning

Clean the instrument regularly to ensure its hygiene and proper operation. Wipe the outer casing with a damp cloth while the instrument is powered off, using soapy water or a neutral detergent.

1.3 Fuse Replacement

When replacing fuses, to prevent electric shock, the power must be turned off and the power plug unplugged.

2 Repair

After purchasing the instrument, keep the warranty card. Send it along with the instrument to the manufacturer or distributor for warranty coverage. The main unit is covered by a one-year free warranty from the date of purchase for non-human-caused quality issues (Note: Warranty is based on the purchase receipt and warranty card; this warranty only covers the main unit and does not include accessories or other consumables that require regular replacement). After the warranty period, please contact the manufacturer directly for after-sales service. Repairs must be made through our sales department or distributor; do not attempt repairs yourself. Unauthorized opening of the seal will void the warranty.

Manufacturer's Responsibility

- Products manufactured should conform to product standards. If any do not conform, our company is responsible for replacement or return.
- Our company assumes corresponding responsibility for personal injury or property damage caused by product manufacturing defects, except in the following circumstances:
- Purchasers or operators fail to use the product according to the instruction manual or safety precautions and warnings.
- Violations of regulations regarding product transportation, installation, use, maintenance, and storage cause product, personal injury, or property damage.
- Our company provides lifetime maintenance and technical services for our products. Manufacturing quality issues occurring within the specified warranty period will be repaired free of charge.
- When replacement parts are needed, original parts must be purchased from our company; otherwise, any adverse consequences will be borne by the user.

Storage and Transportation Management

- Storage: Instruments should be stored in their original packaging boxes in a well-ventilated indoor environment. The packaging boxes should be elevated. The ambient temperature should be -10°C to 40°C, and the relative humidity should not exceed 93%. They must not be placed near harmful gases, flammable or explosive substances, or corrosive gases.
- Transportation: Under packaged conditions, instruments are suitable for road, rail, air, and water transportation. During loading, unloading, and transportation, severe vibration and impact should be prevented. They must not be exposed to moisture and must not be mixed or transported with flammable or corrosive substances. Specific requirements are as specified in the purchase contract.

VII

Warranty Terms

The warranty period for this instrument is 1 year (Note: excluding damage caused by human error). Instruments exceeding their service life must be inspected and approved by an engineer before continued use. Regular performance testing by an engineer is required during subsequent use. Scrapped instruments and accessories exceeding their service life should be disposed of according to local regulations. Subject to the sales contract.

Repair Record Form			
Product Name		Purchase Date	
Distributor Name		Telephone	
Distributor Address			
Customer Name		Telephone	
Customer Address			
Repair Date	Cause of Fault	Repair Content	